Technical Specification Sheet

JAN 2021, Vol.1

DOBU Mask Particulate Respirator

The NOISH approved Dobu Mask 201 Series filtering facepiece respirators are designed to help provide respiratory protection against certain airborne contaminants. The respirator shall be used in accordance with the instructions and limitations as well as applicable safety and health regulations.



INSTRUCTIONS

- ✓ Single Use Only
- Use for solid particulates and mists in concentrations not exceeding 10X PEL/OEL
- This respirator helps reduce exposures to certain airborne contaminants
- ✓ Read User Instructions carefully before use
- Only use the manufacturer's replacement parts as specified

Key Features

- NIOSH approved N95 rating
- High efficiency HEPA Filter, 3 Layers
- Low Exhaust/Inhalation Resistance
- Double elasticated straps
- Light weight (<0.2oz), 2D Folded Type
- Dimension: H. 6.14" (156mm) x W. 7.95" (202mm)
- 100% made in Korea including raw materials provided by qualified supplier's

Approvals and Standards

- NIOSH approval No.: TC-84A-7851
- NIOSH 42 CFR 84 Class
 N95 requirements for a min. 95% filtration efficiency
- Cautions and Limitations: ABCJMNOP
- Assigned Protection Factor (APF 10) per US OSHA and MSHA

State Specific Contract information

- State of California
- State-wide Contract Number: 1-20-65-32p
- Mask, Respirator and Surgical 95 Non-Vented (Standard/Bulk Pricing)



LIMITATIONS

- ✓ Do NOT use in atmospheres containing less than 19.5% oxygen.
- ✓ Do NOT use for gases and vapors, oil aerosols, asbestos, arsenic, cadmium lead, &c US OSHA has separate substance-specific standards in the Exposure Limits Table
- Do NOT substitute, alter, add, or remove parts.
- ✓ Immediately leave contaminated area
- ✓ Do NOT use as surgical masks

Warning - Misuse may result in sickness or death

Before use, carefully read the User Instructions provided as a part of the product packaging.

For correct use, contact supervisor with the User instructions, or call Customer Service Dept. INFOMOVE INT., INC. as below.

Manufactured by



DOBU Life Tech Co., Ltd 38-3 Hoean-daeo, Maesan-ri, Opo-eup, Gwanju-si, Gyeonggi-do, Republic of Korea Tel (+82) 31-765-9311 www.dobumask.com









Shelf Life and Storage

- 3 years from the date of manufacture
- Use by date on box in YYYY.MM.DD format
- Store respirators in the original packaging, Keep the respirators away from contaminated areas, dust, direct exposure to sunlight, extreme temperatures, excessive moisture, and damaging chemicals
- Store in temperatures between 4°F (-20°C) and +86°F (+30°C) and never exceeding 80% RH. Assigned Protection Factor (APF 10) per US OSHA
- Always verify that its seal before use

Packagings

500 masks per Box (Individually sealed masks)

■ 14,000 masks (28 Boxes) per Pallet







For More Information

Technical Assistance / Customer Service:
Tel 1-714-400-2075
Fax1-714-683-0460
cs@infomove.net
INFOMOVE INTERNATIONAL, INC.
18218 McDurmott E STE G IRVINE, CA 92614
Hours of Operation: M-F 8:00 am - 5:00PM (PST)
wwww.infomoveppe.com